



# Platinum V.EU

Safe and efficient  
endodontics system



# Platinum V.EU

## 📌 A fine-tuned instrument for various clinical cases

### An inspiring selection

Offers a range of files to effectively master canal anatomies from simple to complex. Start with an intuitive 3-file basic sequence, then get inspired by the wide range of files. Choose the ones to compose your own sequence and pursue your own therapeutic approach.

#### THE BASIC SEQUENCE Choose taper according to canal anatomy.

- If the canal anatomy requires further shaping, use larger sizes after reaching working length with 30.04 or 25.06.
- Choose Taper .04 for narrow and highly curved canals, including apical curvature.
- Choose Taper .06 for the majority of clinical cases.




Platinum V.EU: K-File	17/12vt	K-File	15/04vt	20/05vt	25/04vt	30/04vt	25/06vt
Speed :	N/A	300	N/A	350	350	350	350
Torque:	N/A	3	N/A	2.4	2.4	2.4	2.4

# Safe and efficient endodontics system

Bondent presents its patent of cross section with flat design: the perfect compromise between cutting efficiency and debris removal

## Platinum V.EU

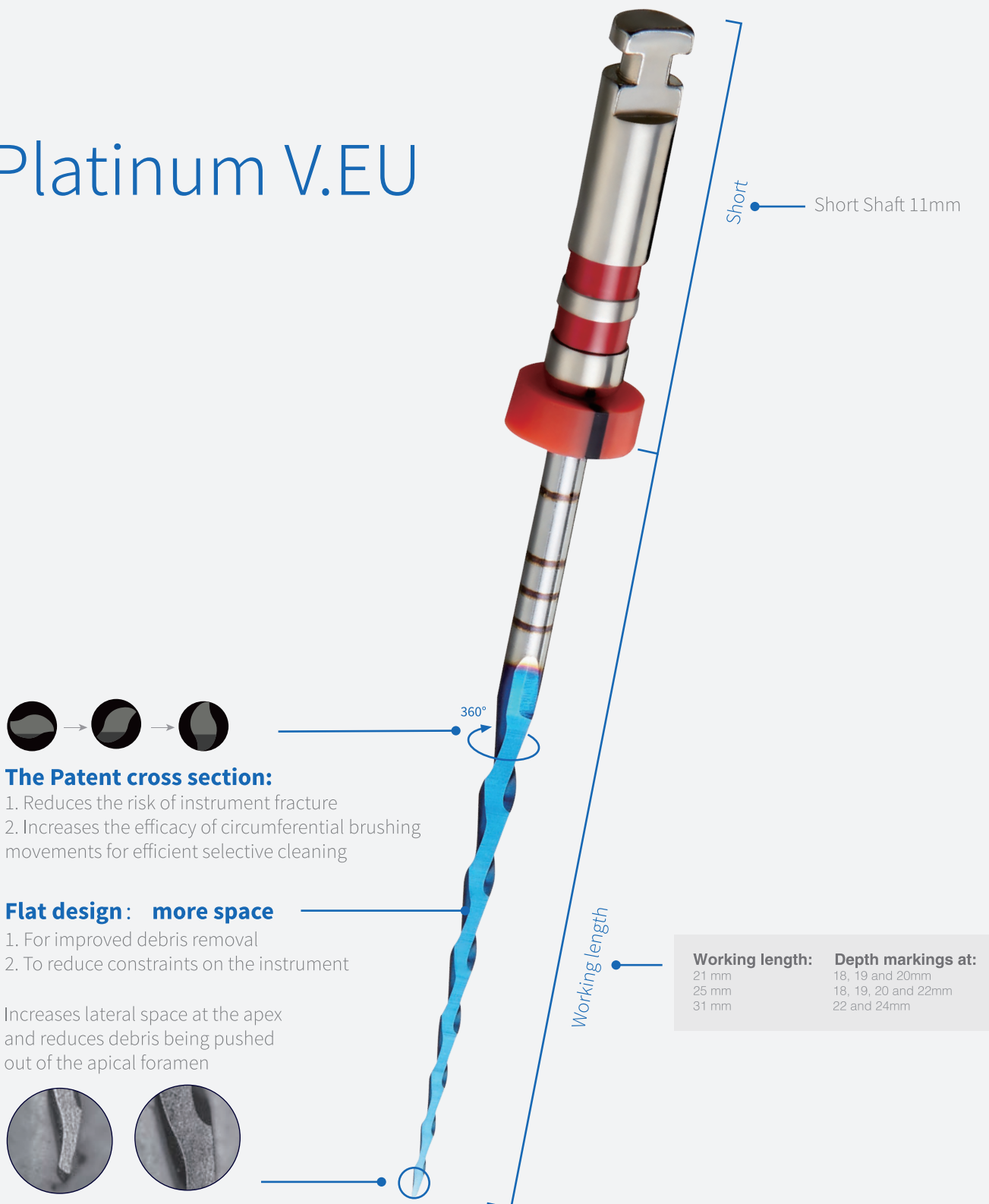

## Product reference




### 17/12vt

19mm PVEU012019017 ×6




### 15/04vt

21mm PVEU004021015 ×6  
25mm PVEU004025015 ×6  
31mm PVEU004031015 ×6

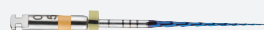


### 20/05vt

21mm PVEU005021020 ×6  
25mm PVEU005025020 ×6  
31mm PVEU005031020 ×6

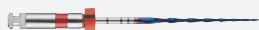


### 25/04vt

21mm PVEU004021025 ×6  
25mm PVEU004025025 ×6  
31mm PVEU004031025 ×6

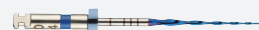

### 30/04vt

21mm PVEU004021030 ×6  
25mm PVEU004025030 ×6  
31mm PVEU004031030 ×6

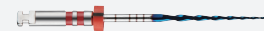

### 25/06vt

21mm PVEU006021025 ×6  
25mm PVEU006025025 ×6  
31mm PVEU006031025 ×6

### Assortment

21mm PVEU000021A01 ×6  
25mm PVEU000025A01 ×6  
31mm PVEU000031A01 ×6




### iRoot Pro

SRCT2000 ○  
SRCT3000 ●  
SRCT2007 ●  
SRCT2008 ●

Cordless Endo Motor  
with integrated Apex Locator

### Standard Gutta Percha Points

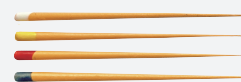

20/04 UDGP004000015 ×60  
25/04 UDGP004000020 ×60  
25/06 UDGP004000025 ×60  
30/04 UDGP004000030 ×60